HANDLEIDING HANDBUCH MANUAL

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Watch the video about adjustment via the QR code.

BACKREST ADJUSTMENT

Loosen the knobs both left and right from the backrest. Adjust the backrest to the desired position and tighten the knobs again.

ARMRESTS SWING AWAY

Slightly lift the armrests to swivel them around. When placing them back, make sure the armrests are in place by pushing them down well enough.

SEAT HEIGHT ADJUSTMENT

Remove the 4 quick release pins from under the seat.

(not applicable to Custom, Blue and Junior XS models)

<https://www.lagooni.com/en/service/manual>

Lift the backrest and seat to a desired position. Replace the quick release pins.

Push the seat max. one pin up on the front of the seat comparing to the rear, to create an angular adjustment. Replace the pins at front and push the rear down (one hole lower) and replace the pins at the rear.

(not applicable to Custom, Blue and Junior XS models)

Unscrew the bolts with an Allen key nr.4 and a spanner nr.10

Watch the video about adjustment via the QR code.

FOOTREST ADJUSTMENT

The mounting of the footrest can also be done with mounting clamps. The adjustment can be done with the same tools.

ANGLE FLIP UP FOOTREST

The flip up footrests have possibility to change the angle. Use an Allan key nr. 5.

ADJUSTMENT FOOTREST OF SWING AWAY LEGREST

The footrest height of the swing-away legrests can be adjusted by loosening the bolts with an Allen key no. 5 and lowering the footrest.


<https://www.lagooni.com/en/service/manual>

Watch the video about adjustment
via the QR code.

The brakes can be locked or
unlocked by pushing or pulling.

(not applicable for the Blue,
Compact and Traveller
with 4 castors).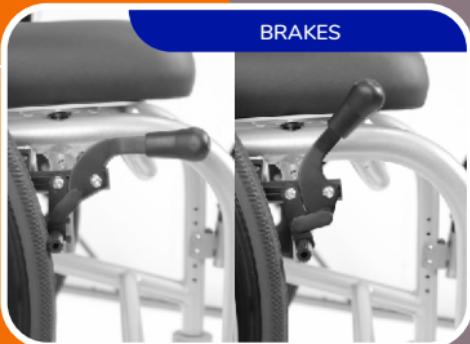

The brakes can be tightened when
needed by using an
Allan key nr. 5.

The anti-tip is mounted under the
seat. Remove or fix it with the pin.

Watch the video about dismantling
via the QR code.

DISMANTLING

Remove the arm-and legrests by lifting them up.

DISMANTLING

Remove the quick release pins and put the seat into the lowest position. Replace the quick release pins.

DISMANTLING

Loosen the knobs of the backrest and remove the quick release pins to turn the backrest.

<https://www.lagooni.com/en/service/manual>

Watch the video about dismantling
via the QR code.

Turn the backrest 180 degrees and make sure the bending faces to the front. Replace the backrest and quick release pins.

DISMANTLING

Fold down the backrest and tighten the knobs.

DISMANTLING

When applicable, remove the rear wheels by pressing the quick release axles.

DISMANTLING

<https://www.lagooni.com/en/service/manual>

Watch the video about dismantling
via the QR code.

DISMANTLING

Remove the 4 small knobs from the legs of the wheelchair

(only applicable for the Compact and Traveller models)

DISMANTLING

Remove the legs from the frame by pulling them out.

(only applicable for the Compact and Traveller models)

PACK IN THE SUITCASE

Open the suitcase and place something inside to protect the wheelchair. Unzip the inside at least halfway.

(only applicable for the Compact and Traveller models)


<https://www.lagooni.com/en/service/manual>

Watch the video about packing
via the QR code.

Place the knobs, axles and other small objects in the side of the suitcase.

PACK IN THE SUITCASE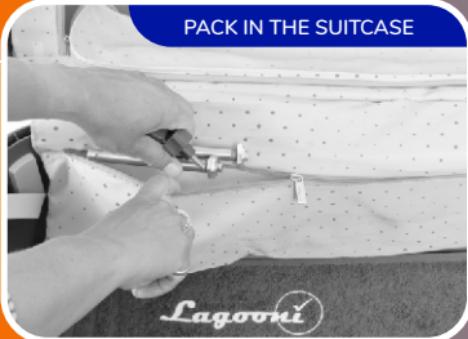

Place the frame with the seat facing down in the suitcase. If the width is 46 cm or more, turn the seat 90 degrees and place these separate from the frame.

PACK IN THE SUITCASE

Place the legs in the suitcase

PACK IN THE SUITCASE

Watch the video about packing
via the QR code.

PACK IN THE SUITCASE

Place the armrests in the suitcase
in between the legs (or in the front
or behind the frame)

PACK IN THE SUITCASE

Place the legrests behind the frame
in the suitcase

PACK IN THE SUITCASE

Place any other objects in the
suitcase (f.e. foldable bucket,
bedpan).

<https://www.lagooni.com/en/service/manual>

When applicable, place the 20" rear wheels in the cover of the suitcase. The wheels should overlap and be placed on one side of the cover.

PACK IN THE SUITCASE

MAINTENANCE

- Remove soap and dirt regularly by using a soft soap. Do not use any caustics like chlorine or alcohol. It is possible to clean the frame with Antikal.
- The seat can be removed to clean the seat and the high-low system properly (not applicable for the Blue, Junior XS and Custom models). • Spray the quick release axles of the wheels with silicone spray once in every three months. • Spray the quick-release pins of the seat and backrest with silicone spray once in every three months.
 - Neoprene and Platilon are comfortable but delicate materials. Be careful with sharp objects like zippers, lifts, transfer aids, etc.
 - Replace the seat over time. Normally, but depending on the frequency and intensity of the use, it can be used for 3 years. After 3 years, Neoprene and Platilon become porous and water soaks into the underlying polyether.

Lagooni BV
Vlotlaan 4a • 2681 RX Monster • The Netherlands
Telephone: (+31) (0)174-281551
info@lagooni.com • www.lagooni.com

WHEELCHAIRS BY
Lagooni
Dutch Quality Product